

OraBudd™

Mouth Ulcer Treatment

Provides relief from the pain and discomfort of oral mucosal ulcers.

The innovative Capsubudd technology functions by targeting damaged tissue, creating a protective seal over the affected area.

This seal aids in the healing process and guards against the infection worsening, after just one application.




OraBudd™ is for the treatment of ulcerations in the tissues of the mouth. It is not intended for the treatment of cold sores, fluid-filled blisters or vesicles in the mouth or on the surface of the lips.

WARNING: Do not use if allergic to sulphur.

INSTRUCTIONS:

Please read carefully

1. Ensure that your hands are clean, break open the plastic tube body along the middle perforation to expose the oral ulcer gel covered cotton tip.
2. Next, roll the cotton tip over the affected area for 5 – 10 seconds, ensuring the entire ulcer is covered. **A burning, stinging sensation is experienced upon application.**
3. Rinse mouth thoroughly with water and spit out.

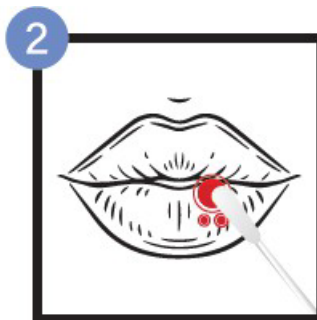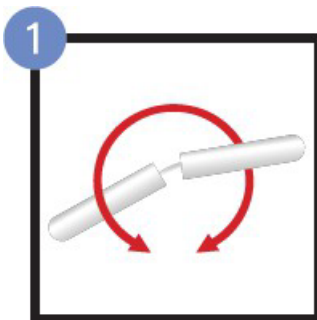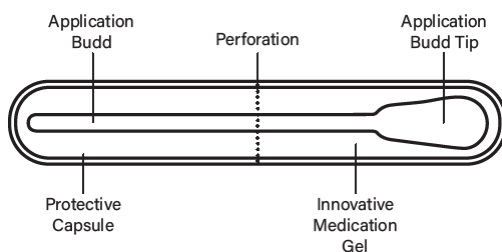

Note: If the ulcer pain remains after rinsing with water, it is an indication that some part of the ulcer was not covered. Repeat application immediately, one more time only, following the directions above. Thereafter do not retreat again.

Important Safety information: Do not use if allergic to materials that contain sulphur in any form. Do not use if you are pregnant, or on children under the age of 12. Avoid contact with eyes. If eye exposure occurs, immediately remove any contact lenses, irrigate eyes for at least 15 minutes with water and seek immediate medical attention.

May cause local irritation or mild swelling when applied.
Avoid contact with clothing as it may discolour clothing.

Keep out of reach and sight of children.
Store at or below 25°C in a dry place.

If used as directed the product cannot be reused.

Ingredients:

Hydrogen Sulphate, Surfactant Sulphonated phenolics, water, colloidal silica, FD&C Red #40.

This pack contains:

2 single-use applicators, each containing a cottonbud covered in OraBudd oral ulcer gel.
5 single-use applicators, each containing a cottonbud covered in OraBudd oral ulcer gel.
Not all pack sizes may be marketed.

Manufactured under license for, and distributed by:

Losaro Service Trust
36 Godlonton Ave
Miramar
Gqeberha
South Africa
6070

Revision date: 11/2024
ORA/IFU/A